



Step 1:

Trace the e-collar template onto the plastic file cover

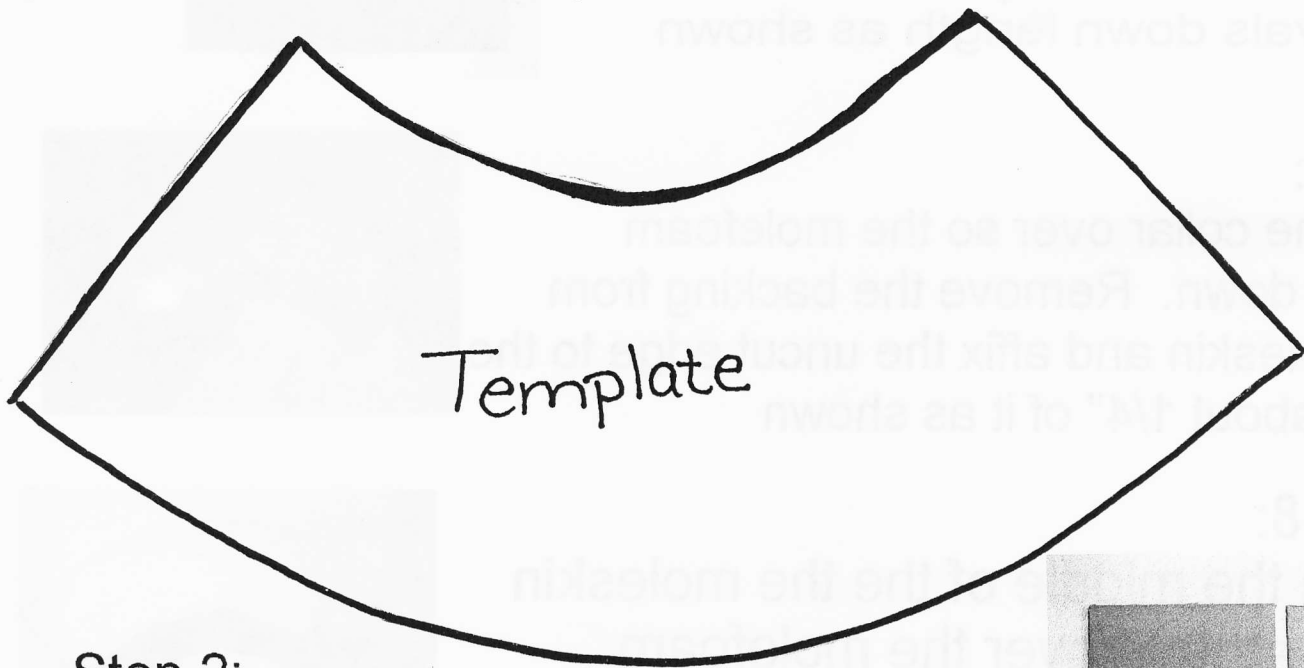


Step 2:

Take 1 sheet of Dr Sholls Molefoam  
Cut 2 strips 1/4" along the long edge.

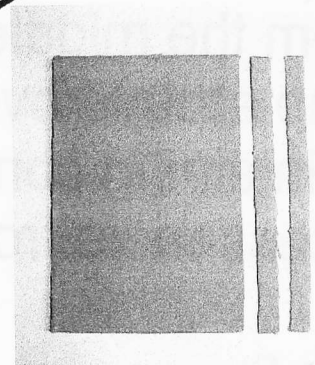


Step 3:

Remove one adhesive strip and stick  
one on top of the other, leaving one  
adhesive strip for later

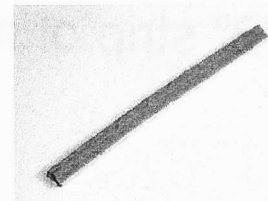

Step 4:

Cut 3/4" off dual layer of molefoam  
Remove the backing and adhere to  
the clear plastic as shown

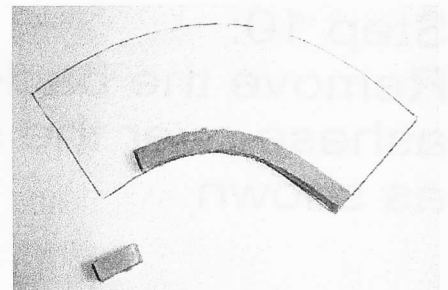

Step 5:

Take 1 sheet of Dr Sholls Molesking  
Cut strip 1 3/8" wide down the long  
edge.

