




Step 6:
Cut slits 1/2" deep at 1/2" intervals down length as shown

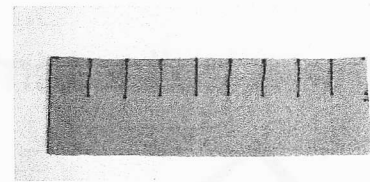


Step 7:
Turn the collar over so the molefoam side is down. Remove the backing from the moleskin and affix the uncut edge to the collar about 1/4" of it as shown

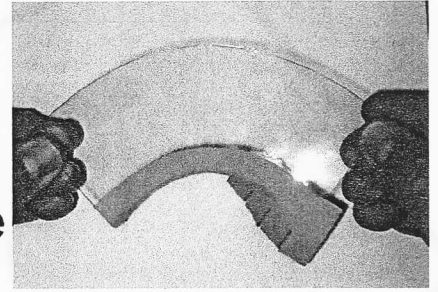


Step 8:
From the middle of the the moleskin wrap around over the molefoam maintaining contours. Trim the excess mole from the edge

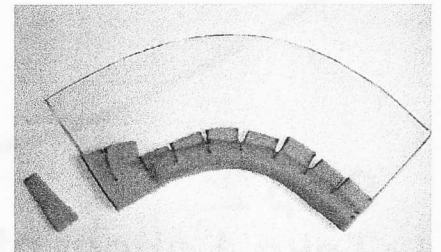

Step 9:
Cut 1/2" strip of moleskin from longest edge.

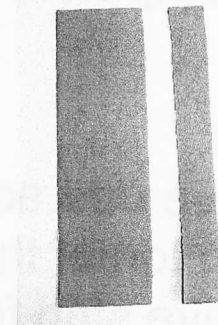

Step 10:
Remove the backing and ashese over the slitted moleskin as shown

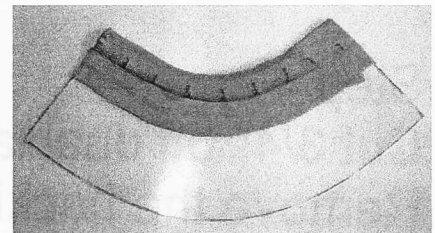

Step 11:
Cut 1 strip 1 3/4" long by 5/8" wide as shown

